


SoftType™ 4.2

On-screen Keyboard

For Windows 95/98/ME/NT/2000/XP

User Manual

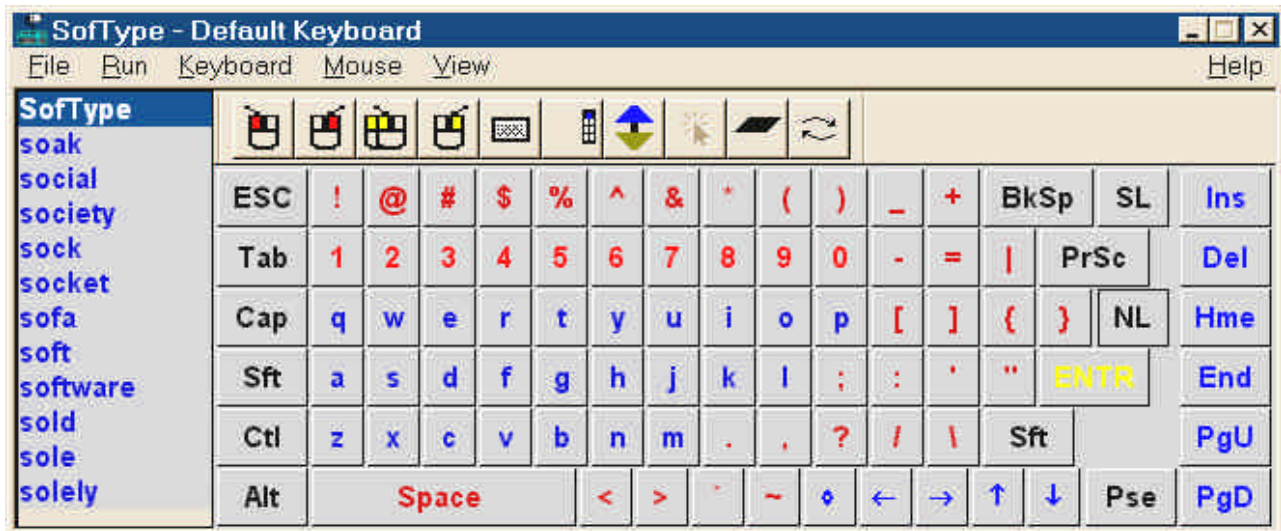

SofType™, Dragger™ and AutoClick™ are trademarks of Origin Instruments Corporation.

HeadMouse is a registered trademark of Origin Instruments Corporation.

SofType 4.2 User Manual Published by:

Origin Instruments Corp.

854 Greenview Drive

Grand Prairie, TX 75050

Tel: 972-606-8740

Fax: 972-606-8741

email: info@orin.com

web: www.orin.com

© 2000 Origin Instruments Corp. All rights reserved.

No part of this publication may be reproduced or transmitted in whole or in part by any means without the written permission from the publisher.

Table of Contents

Introduction	2
Product Support	2
Software Registration	2
Upgrading Note	2
What's New in SofType 4.2	2
Installation	3
Overview	4
Chapter 1 The Keyboard	5
Appearance	5
Keyboard Layouts	9
Custom Keyboard Layouts	13
Moving & Sizing the Keyboard	15
Key Size	16
Colors and Fonts	17
Chapter 2 Dragger & AutoClick	19
The Dragger Toolbar	19
Placement & Orientation:	19
Resizing the Buttons	21
AutoClick	21
Clicking Functions	22
Jitter Box	22
Delay Time	23
AutoClick Sound	23
Other Dragger Settings	23
Repeat Click	23
Chapter 3 Text Entry & Word Prediction	25
Text Entry	25
Word Prediction	25
Add Words to the Word List	26
Edit and Delete Words in a Word List	28
Word Statistics	28
Chapter 4 Abbreviation Expansion & Macros	29
Using Abbreviation Expansion	29
Defining/Editing Abbreviations	29
Using Macros	30
Macro Commands	30
Defining/Editing Macros	30
Macro Tips	31
Example Macros	31
Index	33

Introduction

Congratulations on your purchase of SofType from Origin Instruments Corporation. This manual is a guide to the use and operation of SofType On-screen Keyboard. While SofType does not require extensive training, it is advisable to review this manual before using, to familiarize yourself with this product.

SofType is one of the finest on-screen keyboards available, offering an exceptional alternative to the standard keyboard. Origin Instruments Corporation is committed to providing quality products and exceptional service.

Product Support

Origin Instruments will provide technical support via telephone or email. If the supported product proves defective within 1 year from the date of shipment, Origin Instruments, at its option, will provide a fix for the defective product, will provide a replacement, or will refund the purchase price of the defective product. To obtain service under this support program, the customer must notify Origin Instruments Corp. of the defect before the expiration of the support period.

Contact Information: Origin Instruments Corporation
854 Greenvview Dr
Grand Prairie, TX 75050
Tel: 972-606-8740
Fax: 972-606-8741
email: support@orin.com
Web: www.orin.com

Software Registration

Please complete and return your SofType registration form. Registering your software ensures that you are informed of software updates and new product announcements.

Upgrading Note

Individuals upgrading from a previous version of SofType should be aware that their current SofType settings will not be preserved.

What's New in SofType 4.2

Arrange COMMON Word List: SofType no longer automatically sorts the COMMON word list. The user may now define the order the words are arranged in this list.

Macro Capability: Selecting the Use Macros command from the Keyboard menu enables macro capability. Macros are phrases, commands and other commonly used strings of text that SofType will type for you. Macros can eliminate many keystrokes.

Abbreviation Expansion: Selecting the Use Abbreviations command from the Keyboard menu enables the abbreviation expansion feature. Abbreviations are predefined by the user and allow you to quickly type frequently used word and phrases with a minimum of three clicks.

Installation

When the installation process is complete, SofType 4.2 will be added to Window's Programs menu.

Diskette:

1. Start Microsoft Windows.
2. Insert the SofType 4.2 diskette into your 3.5" floppy drive.
3. Click the Windows "Start" button and select Run...
4. Enter a:setup in the command line text box (or b:setup if your 3.5" floppy disk is in the b: drive).
5. Click OK to begin the installation process.
6. Follow the instructions given by the install program.

CD-Rom: If the installation process does not automatically begin, follow the instructions below.

1. Start Microsoft Windows.
2. Insert the CD-Rom into your CD-Rom drive.
3. Click the Windows "Start" button and select Run...
4. Enter the drive that corresponds to your CD-Rom. For example: m:setup
5. Click OK to begin the installation process.
6. Follow the instructions given by the install program.

Automatically Start SofType: In the install process, you will have the option of having SofType automatically start when Windows starts. If you opted not to, and now want SofType to automatically start, follow the instructions below.

1. Click the Windows Start button and select Settings.
2. Select Taskbar and Start Menu...
3. Select the Start Menu Programs tab.
4. Click on the Add... button, then on the Browse... button.
5. Locate SofType.exe from the Program Files\Origin\SofType42 folder.
6. Click on the Next> button.
7. Locate and double click the StartUp folder.
8. Click Finish.

Overview

SofType is a software utility which replaces the functionality of a standard keyboard with a full featured on-screen keyboard. SofType can be accessed using a mouse or mouse emulator such as the HeadMouse®. SofType 4.2 is compatible with Windows 95/98/ME/NT/2000/XP. SofType works by generating an image of a keyboard on the computer screen. When a key is selected, the character represented by that key is sent to the active Windows application.

SofType has a built-in dwell selection feature called AutoClick™. When AutoClick is enabled, clicking functions are performed by holding the pointer motionless for a programmable length of time.

Dragger is a toolbar integrated into SofType which allows AutoClick and single-switch users to perform all of the common clicking functions of a two-button mouse. Specific buttons on the Dragger toolbar are associated with the common clicking functions. Dragger also has an On/Off button, which enables and disables AutoClick's dwell feature.

Floating the Dragger toolbar is a very functional way to use Dragger. This will allow you to position Dragger on a seldom-used area of the screen. SofType's keyboard can be hidden and still allow the use of Dragger's dwell feature. This is especially useful while using applications which do not require text entry, such as a web browser or card game like Solitaire.

Word Prediction eliminates keystrokes when words are selected from the word list. As you begin to type, a list of words is displayed in the Word Bar, which match the spelling of the word you type. When your word appears in the list, simply select that word. This feature will automatically complete the word and add a space at the end.

The Blackhand Menu is a fast convenient means of accessing SofType's menus and other features. When the hand-shaped mouse pointer passes over the keyboard background, it will turn from white to black, thus the name Blackhand. Performing a left click where the hand is black will open a shortcut window to these features.

Use SofType's Run menu to create shortcuts to your favorite applications for fast access.

SofType offers a variety of keyboard layouts ranging from a very basic keyboard such as the Kidkeys layout to a more comprehensive keyboard such as the AltNate2 layout. It is also possible to create your own keyboard layout by using a text editor to edit an existing layout.

Macro capability and abbreviation expansion have been added to SofType's already extensive list of features. These two new features will make using SofType even easier and more enjoyable by eliminating keystrokes when entering frequently used phrases, commands and other text entries.

New

Chapter 1 The Keyboard

Appearance

SofType offers twelve different keyboard layouts. Most of these layouts have the same basic appearance of a standard keyboard. The Default Keyboard shown in figure 1 consists of seven different areas.

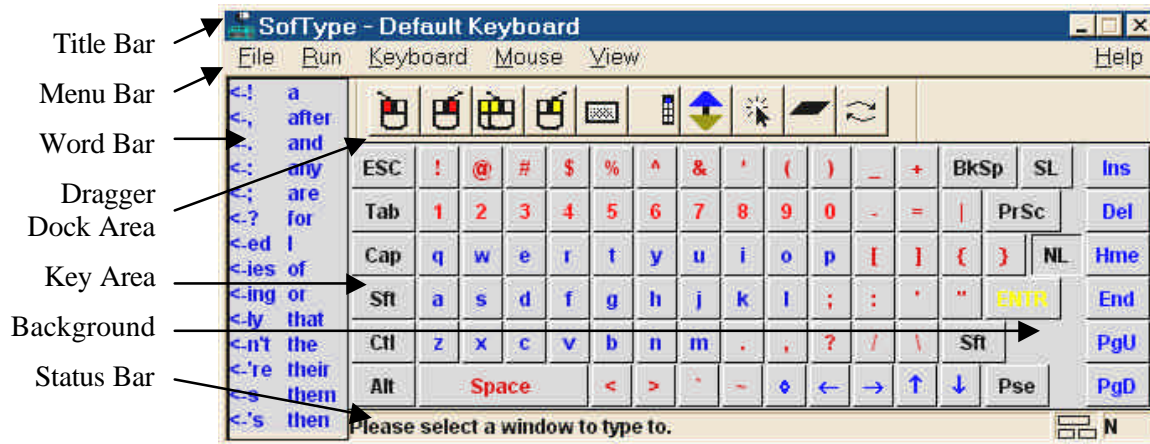


Figure 1: Keyboard Areas

Title Bar: This displays the name of the program, program icon and the name of the keyboard layout that is currently displayed. A Close and Minimize icon are also available from this bar. Should you select the Close SofType icon, a window will open containing the message “Do you really want to quit SofType?” You will have the option of selecting “Yes” or “No.” This is to prevent closing SofType due to inadvertent clicks.

Menu Bar: SofType’s Menu Bar contains five menus to select from to access all of its features and options. These menus are also accessible from the Blackhand Menu discussed later in this chapter.

File: From this menu you can select different keyboard layouts, save and edit words in the word list dictionary and reset the frequency of use statistics. If you wish to close SofType you may select Exit from the File menu [See figure 2] or click on the x in the upper right corner of the SofType window. A dialog box will open asking, “Do you really want to quit SofType?”

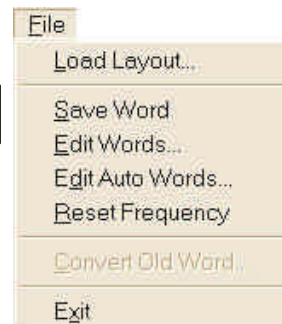


Figure 2: File Menu

Run: This menu allows you to launch your favorite applications from within SofType. Select Edit Run... from the Run menu. This will open the Edit Run Menu dialog box. To add a program to the Run list, click on the Add button. This will open a browse window from which to select a program. After you have made your selection, click Open; the program will now appear in the Run Menu Contents. You may add up to ten programs to the Run Menu. Click OK when you have finished. The programs you have added will now appear in the Run menu. [See figure 3]. If you wish to remove a program, simply select it from the Edit Run Menu dialog box and click on the Delete button.



Figure 3: Run Menu.
Programs added to menu are listed in the lower section of this menu.

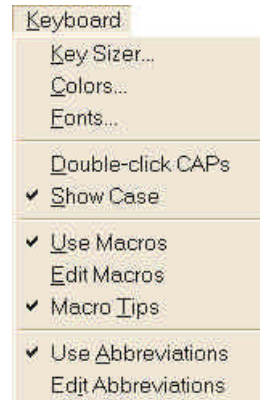


Figure 4: Keyboard Menu

Keyboard: From this menu [See figure 4] you can make adjustments to the keyboard appearance in relation to size, colors and fonts. This is discussed in more detail later in this chapter. The Double-click CAPs and Show Case features are reviewed in Chapter 3. The Abbreviation Expansion and Macro features are also accessible from this menu and are discussed in chapter 4.

Mouse: Selecting Big Cursor from the Mouse menu will increase the size of the pointer while it is positioned over *SofType's* keyboard. Highlighting, when selected, will highlight the keys and words in the word list as the pointer passes over them. [See figure 5]

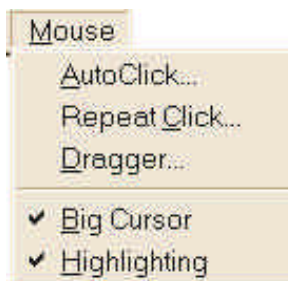


Figure 5: Mouse Menu

View: This menu provides access to the Word Bar Settings dialog box discussed in chapter 3. The Title and Menu Bars can be hidden to create more screen space. You can do this by selecting Title & Menu Bar... from the View menu. [See figure 6] You can use the Blackhand Menu to unhide the Title & Menu Bar. Features, which provide ease in placement of SofType's keyboard, are accessible from the View menu also.

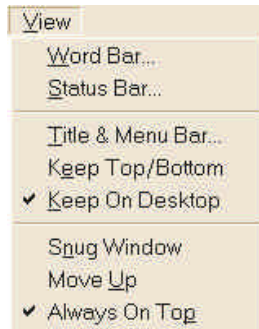


Figure 6: View Menu

Help: The Help menu is located on the right side of the Menu Bar. [See figure 7]



Figure 7: Help Menu

Word Bar: This is a COMMON word list for Word Prediction users. If you are not a Word Prediction user and would like to hide the Word Bar to create more screen space, select Word Bar... from the View menu to open the Word Bar Settings dialog box. Unselect the Words On check box and click OK. Another method is to select Word Bar from the Blackhand Menu.

Dragger Dock Area: This is the area that the Dragger toolbar can be docked in. Four areas are actually available. They are located above, below, to the right or to the left of the Key Area. This area collapses when Dragger is floating.

Key Area: This area contains all of the keyboard keys available for the chosen layout including directional and command keys.

Status Bar: This bar displays your typing status and mouse coordinates and also contains Num Lock and Scroll Lock status. The Status Bar can be positioned above or below the Key Area, or it can be hidden. To reposition or hide this bar, select Status Bar... from the View menu. This will open the Status Bar Settings dialog box. [See figure 8] After making your selections click OK.

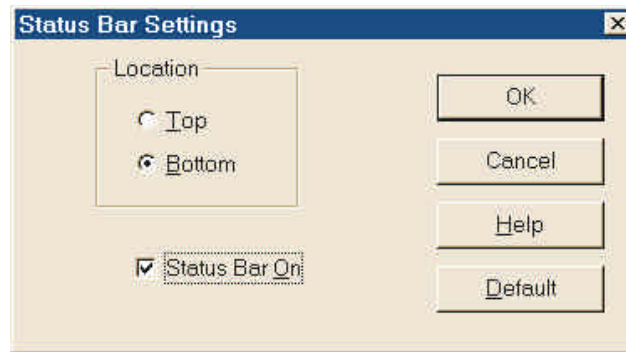


Figure 8: Status Bar Settings dialog box

Background: Clicking on the background within the Key Area will reveal the Blackhand Menu. [See figure 9a] The Blackhand menu is an alternative means of accessing SofType's menus and other features. When the hand-shaped mouse pointer passes over the Key Area background, it will turn from white to black, thus the name Blackhand. [See figure 9b]

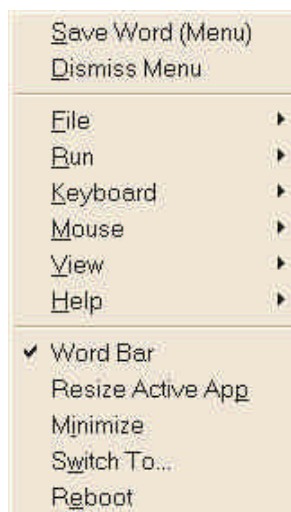


Figure 9a: Blackhand Menu



Figure 9b: Blackhand

Other features accessible from the Blackhand menu are reviewed below. Some of these menus are also accessible from SofType's Menu Bar and have been reviewed earlier in this chapter.

File: Accessible from SofType's Menu Bar and Blackhand menu.

Run: Accessible from SofType's Menu Bar and Blackhand menu.

Keyboard: Accessible from SofType's Menu Bar and Blackhand menu.

Mouse: Accessible from SofType's Menu Bar and Blackhand menu.

View: Accessible from SofType's Menu Bar and Blackhand menu.

Help: Accessible from SofType's Menu Bar and Blackhand menu.

Save Word: Accessible from the Blackhand menu. In the Blackhand menu this option will appear only when you have typed a word that is not already in the word list dictionary. The word that will be added to the dictionary is displayed in parentheses next to this option. [See figure 9a]

Dismiss Menu: Accessible from the Blackhand menu. Closes the Blackhand menu.

Word Bar: Accessible from the Blackhand menu. Selecting this will hide or show the Word Bar.

Resize Active App: Accessible from the Blackhand menu. This feature will shrink the size of the active application and adjust its location to be just above or below SofType's keyboard. This will allow the user to enter text into an application without the flow of text disappearing behind the keyboard. To restore your application to a full screen size, simply click on the Maximize button located in the top right corner of the application.

Minimize: Accessible from the Blackhand menu. Minimizes the *SofType* window. It is important to note that minimizing *SofType* is not the same as hiding the keyboard. Hiding the keyboard is discussed later in this chapter.

Switch To...: Accessible from the Blackhand menu. Opens a Task list, which allows you to quickly move between open applications.

Reboot: Accessible from the Blackhand menu. Will allow you to restart Windows, reboot system, or log off then log on as different user. Choose Reboot to select from three ways to restart your computer: Reboot System closes all running programs and restarts your computer. Restart Windows closes all running programs and restarts Windows. (This option is disabled in Windows NT.) Log Off Windows closes all running programs, logs the current user off of Windows, and then allows you to log onto Windows as a different user.

Note: When you select one of the above options, SofType attempts to close all running programs. If the attempt fails to close any program SofType will not close.

Keyboard Layouts

As mentioned earlier in this chapter, SofType offers a choice 12 different keyboard layouts; in addition, it is possible to edit an existing layout using a text editor. To choose a keyboard layout, select Load Layout... from the File menu. Select the desired keyboard layout and click Open.

The chart on the next page shows the keyboard layout file name, number of key rows in the layout and the general placement of keys. A blank cell indicates that key or set of keys is not available within that layout. This chart is useful in identifying differences between similar layouts and locating keyboards with specific features such as function keys.

Figures of seven of these layouts follow the chart for visual perspective. Variations of the same keyboard are not shown. [See figures 10a – 10g]

Alphabetic layouts have the letter keys arranged in alphabetical order.

Chubon layout arranges the keys by frequency of use with the most frequently used keys clustered toward the center.

QWERTY layouts refer to a standard keyboard layout.

Vertical layout is arranged with the letter keys in a vertical arrangement.

KidKeys layouts contain only the basic keys; these layouts were designed with young children in mind.

Key code:

L: Left

UL: Upper Left

LL: Lower Left

CL: Center Left

R: Right

UR: Upper Right

LR: Lower Right

CR: Center Right

U: Upper

Keyboard File Name	Alpha/Num Layout	F Keys	Esc	NL	SL	Print Screen	Arrow Keys	Home Keys*	Num Pad	Macro
AlphaKey.kys (6 rows)	Alphabetic (3 rows)	U	UL	UR	UR	CR	LR	UR	R	CR
AlphaKy.kys (6 rows)	Alphabetic (3 rows)	U	UL	UR	UR	CR	UR	UR	R	CR
AlphaKy2.kys (6 rows)	Alphabetic (3 rows)	U	UL	UR	UR	CR	LR	UR	R	CR
AltNate1.kys (6 rows)	Alphabetic (2 rows)	R	UR	CR	CR	LR	LL	LC		UC
AltNate2.kys (6 rows)	Alphabetic (4 rows)	R	LR	LR	LR	CR	UC	UR		LL
Chubon.kys (6 rows)	Chubon (frequency of use)		UL	CR	UR	UR	LR	R		LR
Default.kys (6 rows)	QWERTY		UL		CR	UR	LR	R		LC
QWERTY.kys (6 rows)	QWERTY	L	UL	CR	UR	UR	LR	UR	R	CR
KidKeys.kys (4 rows)	Alphabetic (2 rows)									
KidKeys2.kys (4 rows)	Alphabetic (2 rows)								R	
NumPad.kys	None						LC	U	R	
Vertical.kys	Vertical		UL				UR	U	CR	LC

*Home Keys refer to the following keys: Home, End, Insert, Delete, Page Up, Page Down

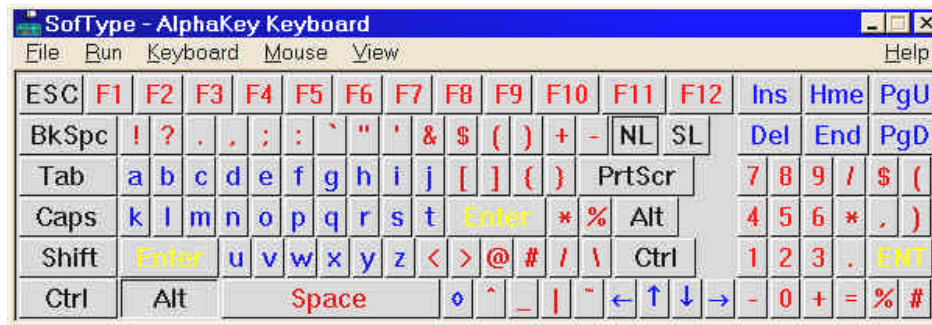


Figure 10a: AlphaKey Keyboard

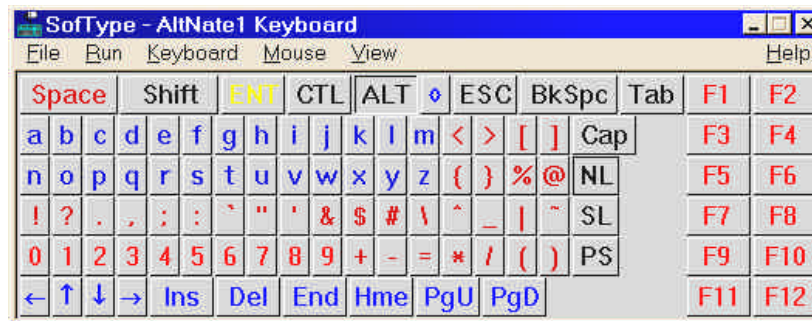


Figure 10b: AltNate Keyboard



Figure 10c: Chubon Keyboard

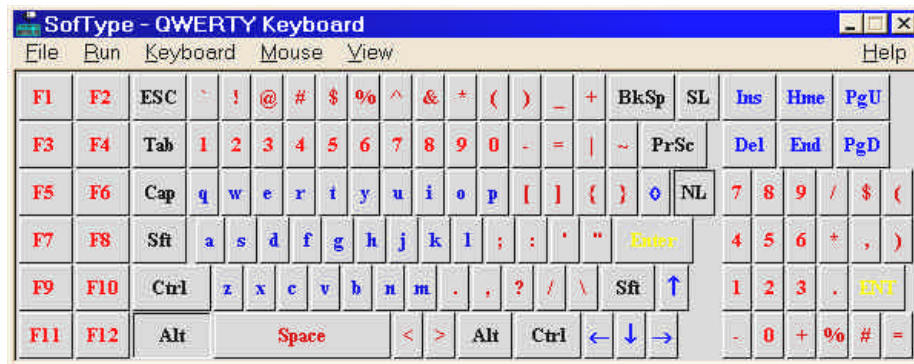


Figure 10d: QWERTY Keyboard

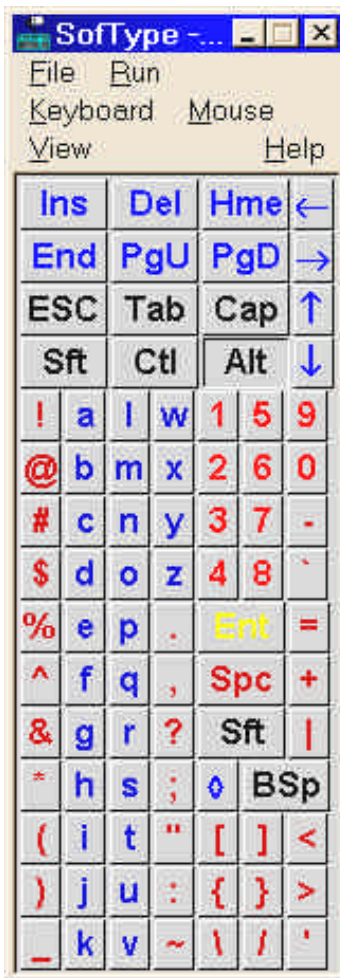


Figure 10f: Vertical Keyboard

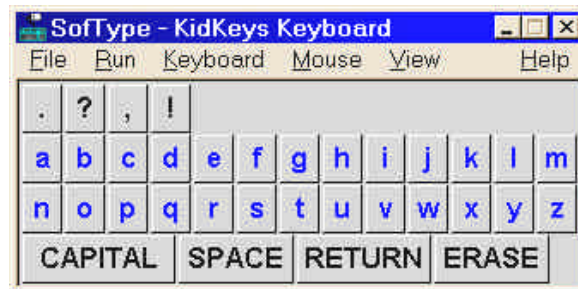


Figure 10e: KidKeys Keyboard

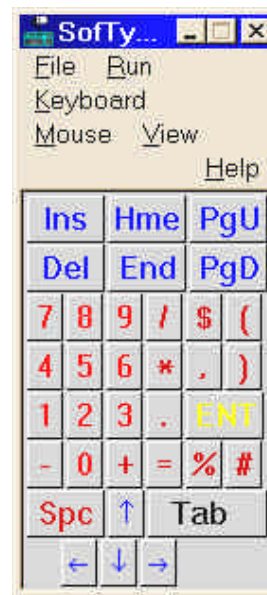


Figure 10g: NumPad Keyboard

Custom Keyboard Layouts

CAUTION: If you cannot use your computer without SofType you may need someone who can use the computer's keyboard if you get in trouble.

The easiest way to create your own SofType keyboard is to edit an existing SofType keyboard. Follow these steps to edit an existing keyboard file:

1. Using the Options menu item **KEYBOARD LAYOUT**, determine which existing layout you want to edit.
2. Using Microsoft Window's Explorer, Make a backup of the .KYS file you wish to edit.
3. Load Microsoft Windows "Notepad" application, or other text editor.
4. Load the .KYS file you wish to edit into Notepad.

What you see with Notepad:

Comments: Lines beginning with a semicolon (;) are comment lines. A comment can contain anything you wish.

Note: A comment line must have a ; as the very first character of the line.

Key Line: A key line looks like this:

1,27,27,0,6,"ESC"

Key lines contain information that SofType uses to represent a key to you and to other application. Key Lines contain 6 fields as follows:

```
Scan Code-----1,
Virtual Key Code-----27,
ASCII Code-----27,
Extended Code-----0,
Color Code-----6,
Text that appears on the key face-----"ESC"
```

Unless you understand exactly what fields 1-5 are, DO NOT change them!

You may change the text of the key (field 6) if you desire. Any text for a key must appear between two quotes. Text for a quote (") would be ""

Please note that the direction keys in the .KYS file do not appear as arrows. They appear as follows: left=⌵, up=⌴, right=⌶, down=⌷.


SPACE: SPACE is a command to SofType to add a space or gap between keys. The gap associated with SofType's SPACE command is one pixel. You may use more than one consecutive SPACE command. SPACE only adds a gap in the horizontal direction. There is no command for a vertical gap.

NEWLINE: NEWLINE is a command for SofType to begin a new line (i.e., row) of keys. A new line of keys begins at the left edge of SofType's window or the left edge of the current "block".

NEWBLOCK: NEWBLOCK is a command for SofType to create a new block (i.e., section) of keys. After a NEWBLOCK command is encountered, SofType will align the new key block to the right of the previous block's widest line of keys, so that the top line

of the new block aligns with the top line of the previous block. Please refer to the following symbolic representation of a .KYS file and the resulting "layout".

MACROKEY: MACROKEY is used for placing SofType's macro key within a keyboard layout. Only one MACROKEY command should appear in a layout.

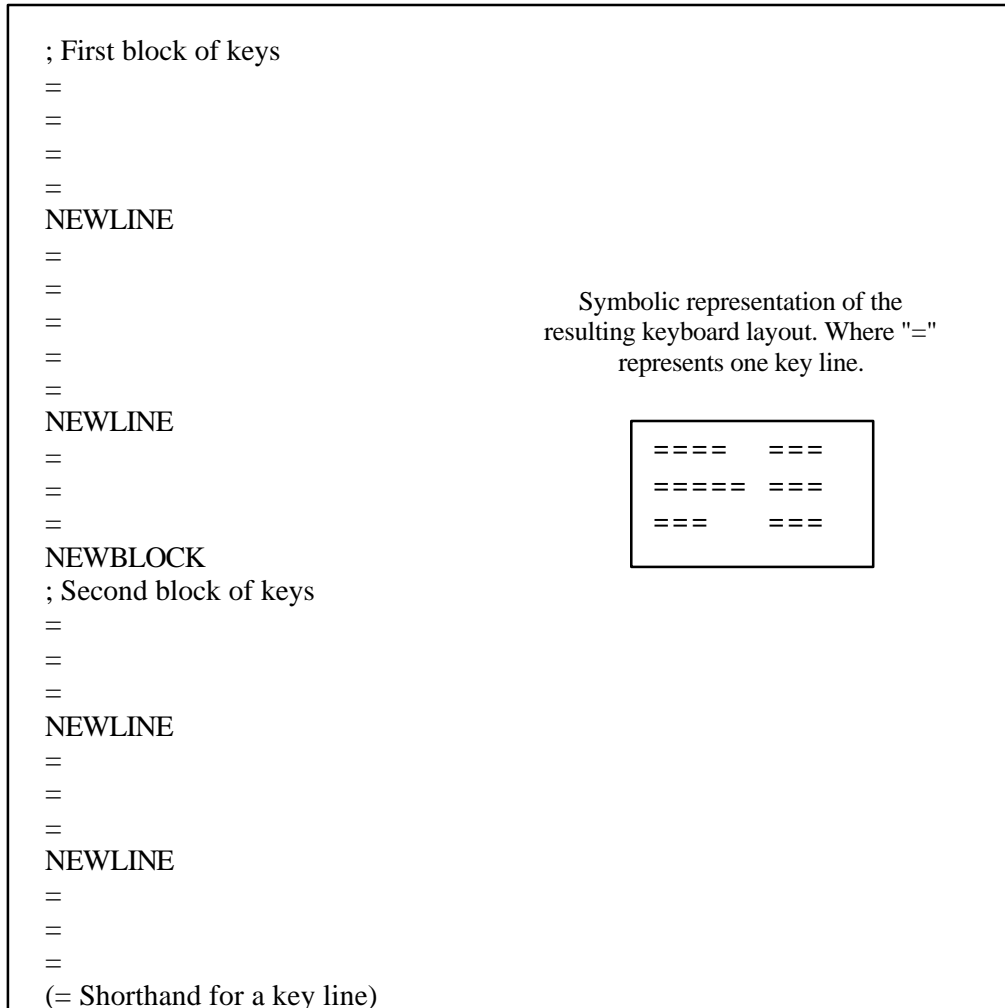

Editing

Select a key line and Cut & Paste to move a key, or Copy & paste to copy a key. Be sure you select all the data on a key line, or range of key lines, before you cut or copy and paste.

Add SPACES, NEWLINEs and NEWBLOCKs as needed.

You may comment out a key line to "hide" a key from SofType.

After the .KYS file has been changed save it and load the file into SofType to see the effects. Several iterations may be required to get the desired result.

Tip: Before editing a .KYS file, load it into SofType and change the size and font of the keys to your preferences and needs. Changing these after you design a layout may change the alignment on certain keys in you layout.

Moving & Sizing the Keyboard

SofType's keyboard can be relocated and resized by a variety of methods to best suit the needs of the user. You should begin by making sure that Always on Top has been selected from the View menu to prevent the keyboard from disappearing behind the application you are working in. You can hide the keyboard while working in applications that don't require text entry or searching the Internet. To do so, click on the hide/show button located on the Dragger toolbar.

Location: It is possible to drag the keyboard by clicking and holding on SofType's Title Bar while moving the mouse pointer. An outline of the keyboard will be visible to assist in optimal placement. (For some users, the window will visibly move as you drag.)

To quickly move the keyboard to the top or bottom of the computer screen, click on the Move SofType button on the Dragger toolbar. This feature is especially useful when the text you are entering becomes hidden by SofType.



Move SofType Button

Another method of sending the keyboard directly to the top or bottom of the computer screen is to select Move Down (Up) from the View menu. If you would like for the keyboard to remain at either the top or bottom of the screen, selecting Keeep Top/Bottom from the View menu will allow the keyboard to be positioned only in these two locations.

To prevent the keyboard from moving beyond the edges of the computer screen where it would only be partially visible, select Keeep on Desktop from the View menu. [See figure 11]

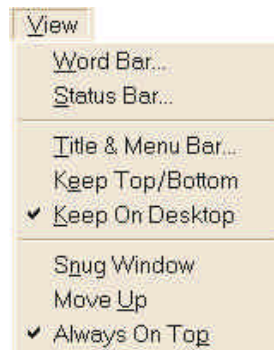



Figure 11: View Menu

Size: The most convenient way to adjust the size of the keyboard is to position the mouse pointer over a corner or an edge of the SofType window, when the pointer changes to a 2-directional arrow, \longleftrightarrow click and hold to drag the keyboard to the desired size.

To quickly scale down the size of the keyboard, SofType allows you to hide specific areas of the layout. Hiding the Title, Menu, Status and Word Bars, and floating Dragger will leave you with only the Key Area using screen space. You have the option to hide some or all of these areas as needed. You will still be able to drag the keyboard to reposition it by holding the pointer over the center of any edge of the keyboard. When the

appearance of the pointer changes to a 4-directional arrow,  drag the keyboard to the desired location.

Hide Title & Menu Bar: There are two ways to hide the Title & Menu Bars. One way is to click on the Shrink/Grow button on the *Dragger* toolbar. You will still have access to all of *SofType*'s menus via the Blackhand Menu. To show these bars again simply click on the Shrink/Grow button again. Another way is to select Title & Menu Bar... from the View menu. This will open a Title & Menu Toggle window. In this window you have the option to keep *SofType* in this state for this session only or every time you run *SofType*. After you have made your selection, click OK.

Hide Status Bar: Select Status Bar... from the View menu. This will open the Status Bar Setting dialog box. Unselect the Status Bar On check box and click OK.

Hide Word Bar: Select Word Bar... from the View menu. This will open the Word Bar Settings dialog box. Unselect the Words On check box and click OK. This option is also available from the Blackhand Menu. It is important to note that hiding the Word Bar will make your Word Prediction inaccessible. Another option is to locate the Word Bar above or below the Key Area. To do this, open the Word Bar Settings dialog box. Select the desired Location toggle, Top, Bottom, Left or Right. Click OK after you have made your selection.

Key Size

SofType has a control that allows you to adjust the size of the keys and the gap between the keys. Utilizing this feature will not only affect the size of the keyboard, but is also an excellent stability compensation tool. Increasing the size of the keys and key gap makes it easier to hold the pointer on the desired key when instability is present. To adjust the key and gap size, select Key Sizer... from the Keyboard menu. This will open the Key Size & Gap Adjust dialog box. [See figure 12] The horizontal and vertical key size and key gap values are adjusted by changing the x and y-axis values respectively. Click OK after you have made your adjustments.

Note: This will not affect the font size on the key face.

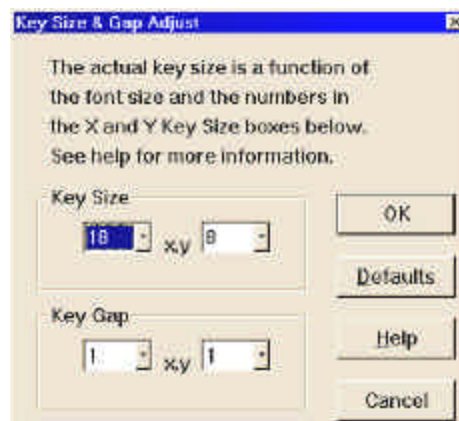


Figure 12: Key Adjust dialog box.

Colors and Fonts

SofType provides a means of changing the colors and fonts that are displayed on the keyboard and key face. The Key Area, including the Background and Word Bar, are divided into groups; punctuation, letters, numbers, controls, etc. The colors for each of these groups can be changed to provide better contrast or make particular groups more easily identifiable. The font size, type and style on the key face, in the Word Bar and in the Status Bar can also be changed. Making font adjustments will change the key size in accordance with the adjustments made.

Changing Colors: To adjust the colors select Colors... from the Keyboard menu. This will open the Color Selection dialog box. [See figure 13] On the left of the dialog box you will see a listing of the Color Groups. Select the radio button associated with the color group you would like to adjust. The current color for the chosen group is displayed in the Test area of the dialog box. Use the Red, Green and Blue slider bars to adjust the color. Click OK after you have achieved the desired color.

Changing Fonts: To adjust the font displayed on the key face, select Fonts... from the Keyboard menu. This will open the Font dialog box. From this dialog box, select the font type, style and size. A sample text will appear in the Sample area of the dialog box. Click OK after you have made your selection.

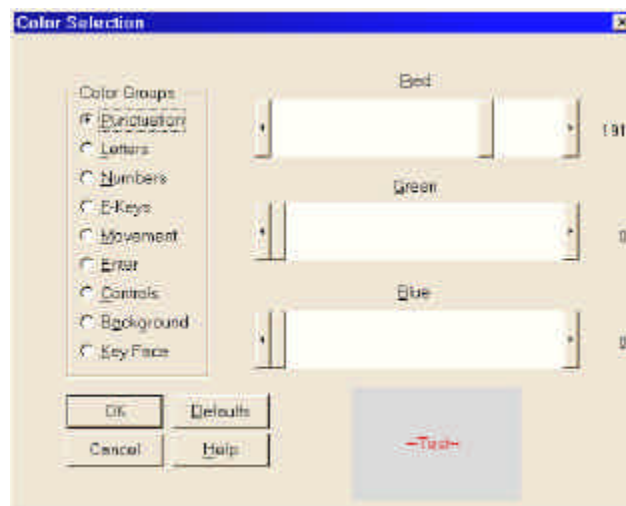


Figure 13: Color Selection dialog box.

Chapter 2 Dragger & AutoClick

The Dragger Toolbar

Dragger is a toolbar integrated into SofType, which allows AutoClick & single-switch users access to all of the common functions of a two-button mouse. Dragger converts a single click into a double click, right click or drag operation.

Dragger consists of ten buttons. [See figure 14] Four of these buttons are associated with clicking functions, one will reconfigure Dragger, one is an ON/OFF toggle for AutoClick users and the remaining four buttons are associated with SofType's keyboard. The use of these buttons will be reviewed in this and later chapters.

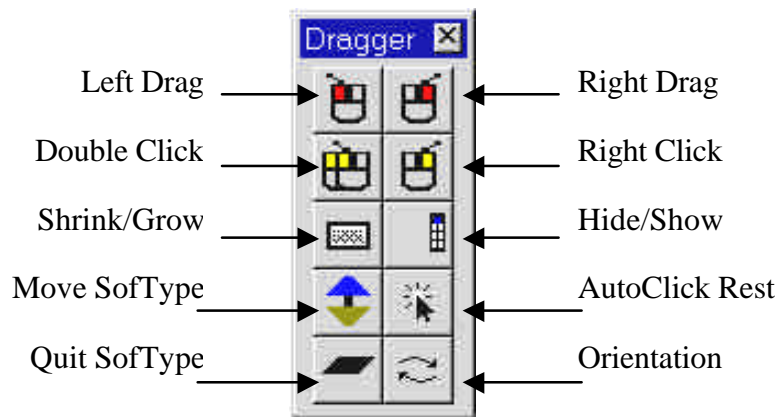


Figure 14: Dragger Toolbar

Placement & Orientation:

Dragger can be docked within SofType's keyboard either above or below or to the right or left of the keyboard. When SofType is first opened, the Dragger toolbar will be docked just above the first row of keys on the keyboard. If the Dragger toolbar is not visible, click on SofType's Mouse menu and select Dragger... Select Dragger On and click OK. Dragger should now be visible. To relocate Dragger, simply drag the toolbar to the desired location. Dragging the toolbar is done by clicking and holding on Dragger's background. The outline will be visible as you drag the toolbar. When you reach one of the docking areas, the outline will become visibly smaller to indicate where to drop the toolbar. [See figure 15]

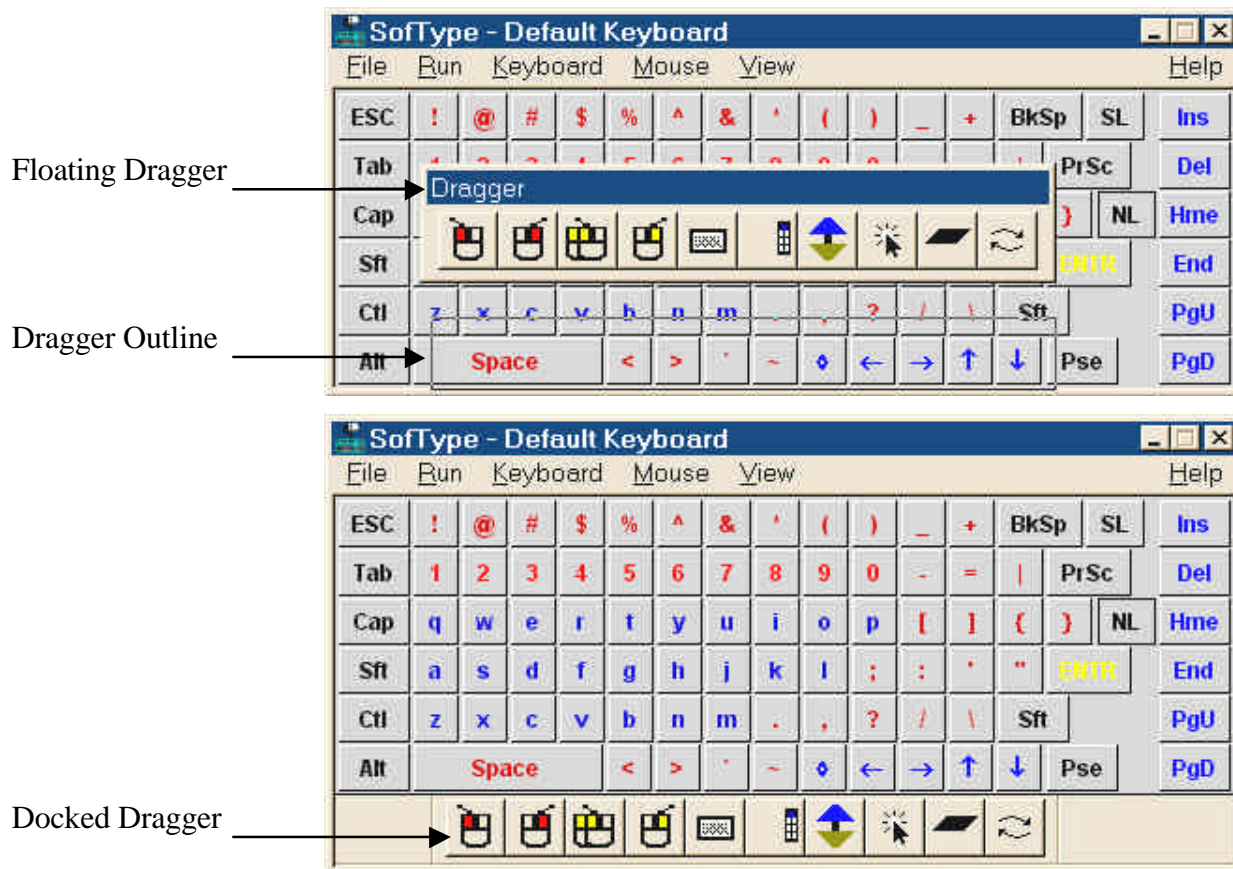


Figure 15: Dragger being moved from float to docked position.

Dragger can also “float”, which means it can be removed from within the keyboard and positioned anywhere on the screen. This feature provides more ease of use as it allows the user to decide the most optimal placement for his/her comfort. For instance, if you are playing Solitaire, you may find that positioning Dragger just below the card stack is most comfortable; however, if you are searching the Internet or typing a letter, you may prefer to move Dragger to the edge of the screen out of the way of your work. You can float Dragger in one of the following ways: Drag it off of the keyboard to the preferred position, double click on Dragger’s background or use the Orientation button as described below. Once Dragger is floating, a title bar appears which can be used to drag the toolbar for more ease in repositioning.

Dragger can be configured either vertically or horizontally, in either a single or a double line. To change the orientation, click on the Dragger Orientation Button [See figure 16] until you reach the desired configuration. Clicking on this button while Dragger is docked in the keyboard will also cause it to float.

Figure 16: Dragger Orientation Button.



Resizing the Buttons

Dragger's buttons can be resized to best suit the needs of the user. To resize the buttons, click on the Mouse menu and select Dragger... The Dragger Settings dialog box will open. [See figure 17] From here you may select from Normal, Large or Larger button sizes. Click OK after you have made your selection.

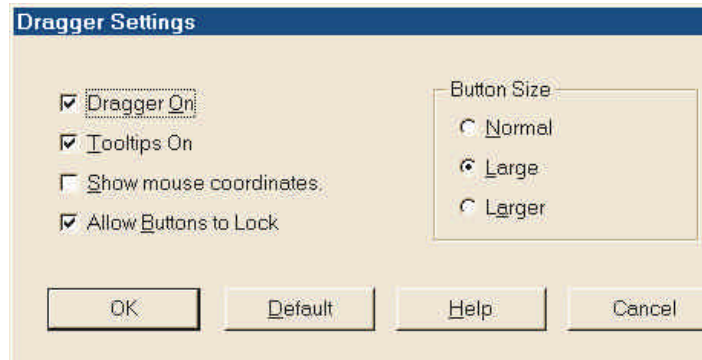


Figure 17: Dragger Settings Dialog Box

AutoClick

AutoClick is a dwell selection technique integrated into SofType. When AutoClick is enabled, clicking functions can be performed by dwelling, or holding the pointer motionless, for a programmable length of time. When the delay time has expired, a click will occur. To enable AutoClick, click on SofType's Mouse menu and select AutoClick... The AutoClick Settings dialog box will open. [See figure 18b] Select the AutoClick On check box and click OK. This will enable AutoClick. You must use the AutoClick On/Off Button on Dragger's toolbar to activate the dwell feature. [See figure 18a] This button will not be available unless AutoClick is enabled in the AutoClick Settings dialog box. To use AutoClick, simply dwell on this On/Off button; the button will have the physical appearance of being depressed and highlighted. To temporarily turn off AutoClick, dwell on the On/Off button again. This is important to avoid sporadic clicking. This button always remains dwell-sensitive allowing the user to work more independently with less outside assistance.



Figure 18a: AutoClick On/Off button.

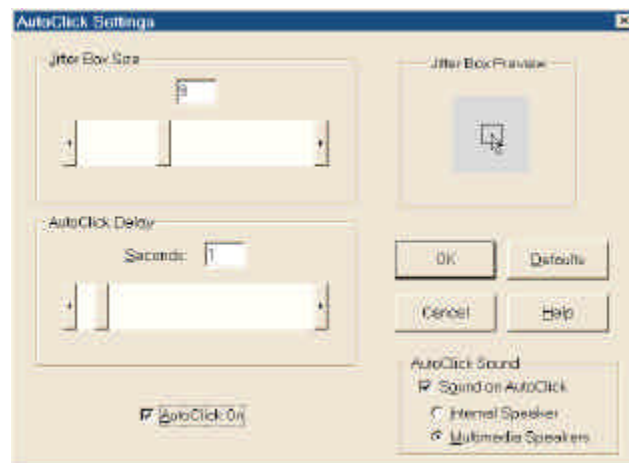


Figure 18b: AutoClick Settings dialog box.

Clicking Functions

When AutoClick is activated, a single left click will occur whenever the pointer is motionless. To perform other clicking functions, drag, double click or right click, you must use the appropriate button on the Dragger toolbar. [See figure19] First you must dwell on the appropriate button, the next place you dwell will perform that function. For instance, to perform a drag you would first dwell on the Left or Right Drag Button; the next place you dwell would begin your drag, and the next place you dwell will end your drag. Double clicks and right clicks are performed in the same manner. AutoClick will automatically revert back to a single left click after the function is performed.

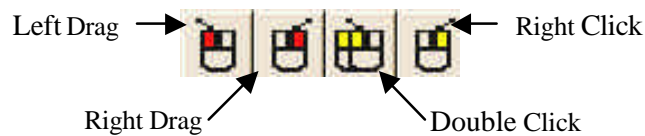


Figure 19: Clicking function buttons.

Dragger's Clicking Function Buttons can also be locked to allow repeated use of that particular function. To lock one of these buttons, dwell on the button a second time. The physical appearance of the button will change to show that it is locked. [See figure 20] When a button is locked, AutoClick will perform that button's function instead of a left click whenever you dwell. To unlock the button simply dwell on it a third time.

If the buttons will not lock, you may need to select the Allow Buttons to Lock option from the Dragger Settings dialog box. To open the Dragger Settings dialog box, click on the Mouse menu and select Dragger... After you have made your selection, click OK.



Figure 20: Left Drag button in locked position.

Jitter Box

The Jitter Box is a stability compensation feature useful for people who use AutoClick but have difficulty holding the pointer motionless for the length of time necessary to perform a click. It is, in actuality, an invisible box that can be adjusted in size. This box determines the size of the area that the pointer needs to be in to be considered motionless. By increasing the size of the Jitter Box, a person who is unable to hold the pointer motionless but can keep the pointer within the Jitter Box area, can then cause AutoClick to perform a click as though the pointer was motionless.

To adjust the size of the Jitter Box, click on SofType's Mouse menu and select AutoClick... The AutoClick Settings dialog box will open. Use the Jitter Box Size slider bar or click on the arrows to make your adjustment. The Jitter Box Preview shows the size of the Jitter Box as it is being adjusted. [See figure 18b] After you have made your adjustment click OK.

Delay Time

The delay time refers to the length of time the pointer must remain motionless before a click will occur. This length of time can be programmed from .2 to 10 seconds. To adjust the delay time click on SofType's Mouse menu and select AutoClick... The AutoClick Settings dialog box will open. Move the AutoClick Delay slider bar or click on the arrows to make your adjustment then click OK.

AutoClick Sound

When enabled, AutoClick provides an auditory feedback as each click occurs. By default, this is a brief "click" however, if a multimedia sound system is installed on your computer, you can change this sound from the Sounds Properties dialog box in the Control Panel of your computer. It is important to note that a very brief sound must be used in order not to interfere with the AutoClick function.

To enable AutoClick Sound, click on SofType's Mouse menu and select AutoClick... The AutoClick Settings dialog box will open. Select the Sound on AutoClick check box and either Internal Speaker or Multimedia Speakers. Click OK.

Other Dragger Settings

Two additional options are available in the Dragger Settings dialog box accessed through the Mouse menu.


Tooltips On: Selecting this option will provide a description of each of *Dragger's* buttons as the mouse pointer passes over it.

Show Mouse Coordinates: Selecting this option will display the x and y positions of the mouse pointer in *SofType's* status bar.

Repeat Click

This feature will allow repetitive clicks by holding down the key just as a standard keyboard would. Select Repeat Click... from the Mouse menu. This will open the Auto Repeat Adjustment dialog box. [See figure 21] Select the Repeat Click On check box to enable this feature. Key Down Delay determines how long you must hold down the mouse button before SofType starts repeating the key. Key Repeat determines how quickly a character will be repeated.

Figure 21: Auto Repeat Adjustment dialog box.

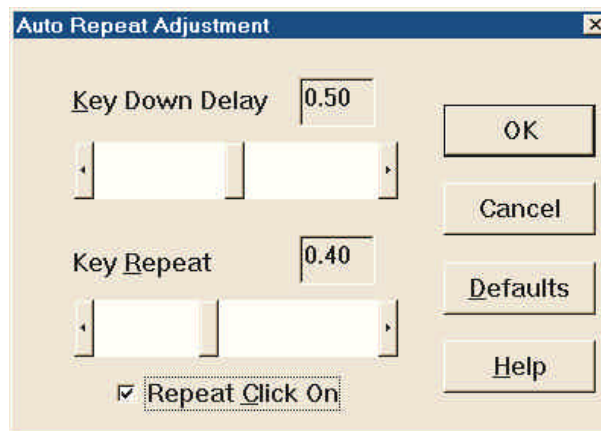

Chapter 3 Text Entry & Word Prediction

Text Entry

To begin entering text you must first select an application to type to just as you would if you were using a conventional keyboard. Place the curser in the location where you wish to begin entering text. This will bring the focus to that particular application. When a key is then selected from SofType's keyboard, the character represented by that key is sent to the active application.

Capitalizing is basically the same as on a standard keyboard. Select the Shift key before entering text, to capitalize the first letter of a word. The Shift key will remain depressed until the first letter has been entered, remaining characters will be entered in lower case. Clicking twice on the SHIFT key will cause it to remain depressed until it is clicked on a third time. Selecting the Cap key will lock caps so that every character will be entered in upper case format. To unlock, click on the Cap key a second time. To display the case format in effect, on the keyboard, upper or lower case, select Show Case from the Keyboard menu. [See figure 22] Another method of capitalization is to select Double-click CAPs from the Keyboard menu. This will allow you to enter a character in upper case simply by double clicking on that character.

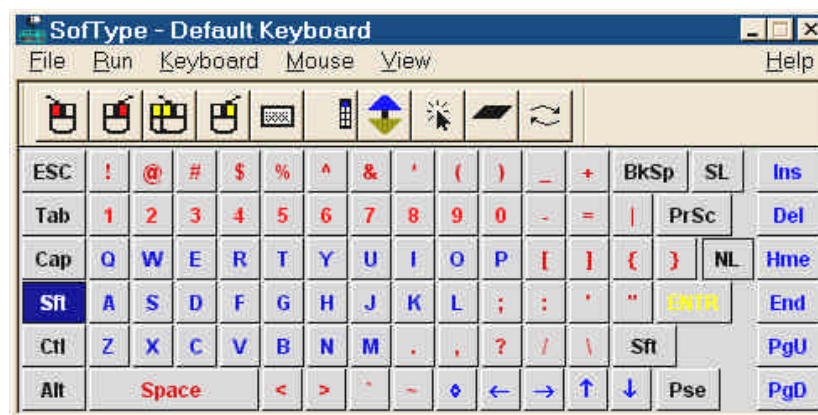


Figure 22: Characters displayed in upper case when Show Case is selected from the Keyboard menu.

Word Prediction

Word Prediction is a feature that reduces keystrokes when words are selected from the word list. As you begin to type, a list of words is displayed in the Word Bar, which match the character sequence you are entering. If the word you are entering appears on the list, selecting that word with a single left click, will enter the remaining characters and a space, in preparation for the next word. You have the option of displaying the words listed alphabetically or by frequency of use. To select the desired order, select Word Bar... from the View menu. This will open the Word Bar Settings dialog box. In the Completion Mode area select **Alphabetic** or **Prediction**.

The word list not only contains words for each letter of the alphabet, it also contains a list of COMMON words. This would include commonly used short words, such as the and that, conjunctions, word endings and punctuation marks. The word endings and punctuations appear in the word list with the symbols "<-" in front of the entry and they have a special feature for removing the space at the end of a word, adding the ending/punctuation and then adding a space. For example, if the word "support_" is selected from the word list, then "<-ed" is selected; the result would be "supported_" (_ signifies a space.)

Word Bar Settings: [See figure 23] As discussed in chapter 1, the Words On toggle will allow you to show or hide the Word Bar. Use the radio buttons in the Location area to determine placement of the Word Bar within the keyboard. The Size area is used to set the number of columns and rows of words that appear in the Word Bar, as well as the number of characters in each word.



Figure 23: Word Bar Setting dialog box.

Add Words to the Word List

Words can be saved manually into the word list, added automatically as they are typed or by entering the new words directly into the Edit Words dialog box located in the File Menu.

Manual Add: If you wish to save a word manually, type the word (do not add a space or select the ENTER key) then select Save Word from the File menu. Entering a space or clicking on the ENTER key after typing a word, resets the word list and makes the Save Word option unavailable. After using this method to add a word to the word list, you will need to select the application you wish to enter text into before you resume typing.

Note: Using the Blackhand Menu to save a word will not transfer focus from your application to SofType.

Automatic Add: SofType has a feature, which allows you to automatically save words that are not included in the word list. Open the Word Bar Settings from the View menu and select Automatically add new words... This will open the Auto Add Settings dialog box [See figure 24]. In this dialog box select Auto add words, then select the desired criteria for adding new words.

Selecting the Show “Edit Auto Words” dialog at SofType exit check box will open a dialog box before SofType closes, from which you may edit/delete any words automatically entered. You can open the Edit Auto Words... dialog box at any time by selecting it from SofType’s File menu.

Selecting Ignore mixed cAsE will prevent words with mixed case, such as SofType, to be automatically added. You may still add these words to the word list using the other methods.

Ignore all CAPS will prevent these words from being automatically added to the word list.

You may also select how many times the word must be typed and the least number of characters it must have before it is automatically added to the word list by entering the desired number in the Type word and Words are fields. After you have made your selections, click OK.

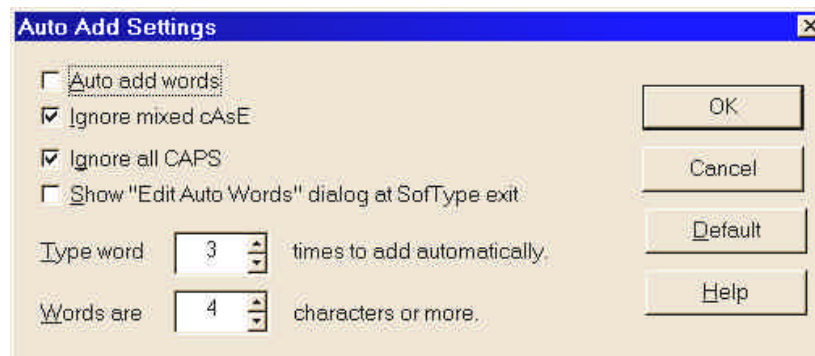


Figure 24: Auto Add Setting dialog box.

Edit Words Menu: In the Edit Words dialog box you can add, remove and edit words as well as re-order words within a word list. Begin by selecting File, then Edit Words. This will open the Edit Words dialog box. [See figure 25]

Use the Move word buttons to re-order the COMMON words as desired. Select the word you want to move then select the Up or Down button to move it within the word list. The word list is grouped alphabetically with the COMMON word group listed in the beginning. Selecting the Next and Previous buttons will display the next/previous letter group. To add a word, begin by selecting the letter group that you want to add the word to. Select the Insert... button; enter the word you wish to add to the list then select OK. It is important to note that the word will be added to the word list that is currently displayed. For example, if the word list for the letter group "A" is displayed and you insert the word "zebra", the word "zebra" will be inserted into the "A" letter group.

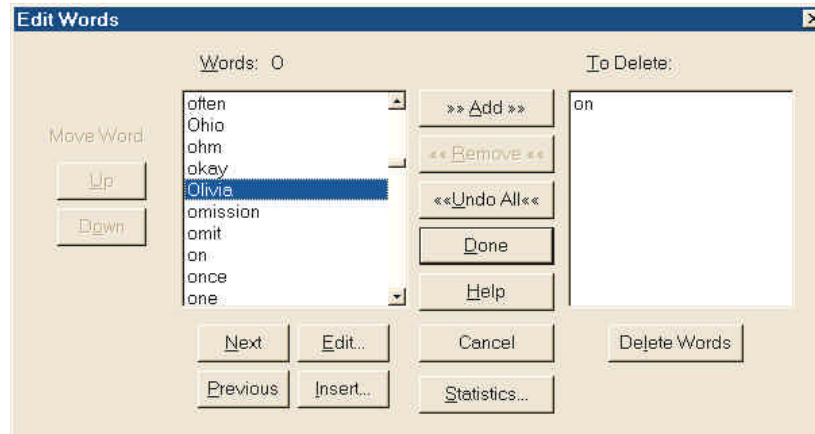


Figure 25: Edit Words dialog box.

Edit and Delete Words in a Word List

To edit a word, select it from the word list and click the Edit button, make the desired correction and click OK. To delete a word, select it from the word list, click Add to move it into the To Delete: list, and then click the Delete Words button. You may move several words to the delete list and delete them all at one time.

Word Statistics

When the Statistics... button is selected from within the Edit Words dialog box, the Word Statistics dialog box opens. This dialog box contains the dictionary word count information. The number of words within each word list group is displayed as well as the total number of words in all word list groups.

Save Prediction Statistics, located in the Word Bar Settings dialog box, is used to determine when and how often you want to save SofType's dictionary file. The settings you choose here are only effective when Completion Mode is set to Prediction. Save Prediction Statistics settings are used to keep Word Prediction statistics up to date in the dictionary file. You can reset the word usage statistics by selecting Reset Frequency from the File menu; this will also alphabetize all the words in the word prediction dictionary.

When the On Exit Only check box is selected, the dictionary file will only be saved when you quit SofType. The Every x Selections field determines the number of selections that must be made from the Word Bar (1-20) before the dictionary file will be saved.

Chapter 4 Abbreviation Expansion & Macros

Using Abbreviation Expansion

Abbreviation Expansion is a feature, which allows the user to define a specific string of characters that SofType would then expand into longer, more complex character strings. As a result, long character sequences can be entered with a minimum of three keystrokes. For example, "ST" could be defined to expand to "SofType On-screen Keyboard". An abbreviation can be 2 to 5 characters and is case sensitive. Using the example above "ST" would be different than "st".

To use this feature you must first enable it by selecting Use Abbreviations from SofType's Keyboard menu. Type the defined abbreviation then the Space or Enter key; using the example above, you would type "ST " to command SofType to first delete "ST", then type, "SofType On-screen Keyboard."

Defining/Editing Abbreviations

To define an abbreviation, select Edit Abbreviations from the Keyboard menu. This will open the Add/Edit Abbreviations dialog box. [See figure 26] Enter the desired abbreviation into the Abbreviation: edit field then enter the expanded text into the Expands to: edit field. Abbreviations must be alpha characters only. Expanded text may be up to 128 characters and may include macro commands. Select the Add To List button to add the new abbreviation to the user-defined list of abbreviations. You may continue to add new abbreviations consecutively to the list. Your abbreviations will be displayed in alphabetical order.

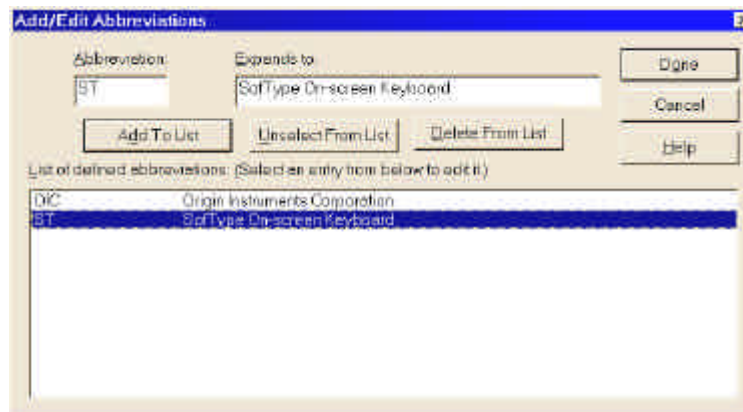


Figure 26: Add/Edit Abbreviations dialog box.

Selecting an abbreviation from the list displays it in the Abbreviation: and Expands to: fields where you may edit the abbreviation or text, or you may delete it by selecting the Delete From List button. If you wish to clear the editing fields, select the Unselect From List button. Selecting this button will only clear the editing fields; the list will not be affected. After you have made the desired additions/changes, select Done. Clicking the Cancel button at anytime will close the dialog box without implementing any changes.

Using Macros

Before defining a macro, you should first understand what it is and how it works. SofType's Macro feature allows you to type phrases and other commonly used strings of text and commands, with only 2 keystrokes. For example, you could define a macro to enter your name and address. This would more quickly allow you to address an envelope or create a letterhead.

To use this feature you must first enable it by selecting Use Macros from SofType's Keyboard menu. Select the Macro Button on the keyboard, [See figure 27] and then select the character key associated with the defined macro. (Note: some keyboard layouts do not contain the Macro Button. Refer to the keyboard layout diagram on page 10.) SofType will then enter the text and commands associated with that macro into your application. For example, the "o" key could be assigned to enter the following text:

Origin Instruments Corp.
854 Greenview Dr
Grand Prairie, TX 75050



Figure 27: Macro Button

Macro Commands

Macro commands are imbedded commands to the macro engine to perform special tasks. These commands consist of a backslash (\) and up to two characters.

- \t- types a Tab key.
- \n- types an Enter key.
- \p- pauses typing for .5 second.
- \ds.....- types the short form of the current date, (02/20/00)
- \dl- types the long form of the current date, (Friday, February 20, 2000)
- \xs.....- types the current time without seconds, (01:30 PM)
- \xm-.....- types the current time without minutes and seconds, (01 PM)
- \xl- types the current time, (01:30:45 PM)
- \xt- types the current time in 24 hour format, (13:30:45)
- \\- types a backslash.

Defining/Editing Macros

To define or edit a macro select Edit Macros from SofType's Keyboard menu. This item will not be available unless Use Macros is selected. Select the key that you want to assign the macro to (if you are editing a macro, select the previously assigned key); this will open the Edit Macro dialog box. [See figure 28] Type the desired string of text and macro commands. You may enter up to 128 characters. The dialog box will display the number of characters in your text as you type. After you have entered the desired text, click OK. You must unselect Edit Macros in order for SofType to execute the defined macro the next time you select it.

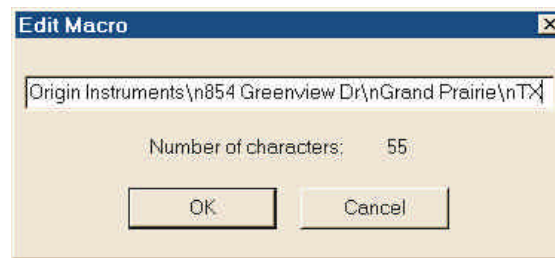


Figure 28: Edit Macro dialog box.

Macros may be defined for all keys except the following: Ctrl, Alt, Shift, Caps, Lock, Num Lock, Scroll Lock or the macro key itself.

Note: When Edit Macros is selected, SofType must be the active application in order to edit/add macros. If SofType is not the active application, anything you type will appear in the active application.

Macro Tips

Marco Tips is a feature that will display a macro description in a tip window when the pointer moves across a defined macro key. This tip window is visible when Macro Tips is selected from the Keyboard menu and the Macro Key is selected or a defined macro key is selected to edit.

Example Macros

The **bolded** examples below will enter the following text by first selecting the Macro Key and then the assigned character key.

username\tpassword\n

To complete a User Name/Password dialog box or web page.

Joe Smith\n720 Highland\nCalm, MS 00000-0000\n

Joe Smith

720 Highland

Calm, MS 00000-0000

Hello everyone! My name is Sally. It's \dl, \xs here.

Hello everyone! My name is Sally. It's Friday, February 20, 1998, 01:30 PM here.

Joe Smith\t720 Highland\tCalm,\tMS\t00000-0000\t(555)-555-1234\n

To complete a typical name/address dialog box.

Index

- A**
- Abbreviation Expansion..... 29
 - AutoClick
 - dwell time..... 23
 - on/off 21
 - settings 21
 - sound 23
- B**
- Blackhand Menu 8
- C**
- Clicking Functions..... 22
 - See AutoClick*
 - repeat click 23
- D**
- Dragger
 - dock 7, 19
 - float 20
 - lock buttons..... 22
 - on/off 19
 - orientation 20
 - settings 23
 - buttons 21
 - toolbar..... 19
- F**
- File Menu 5
- H**
- Help Menu..... 7
 - highlight keys..... 6
- J**
- Jitter Box 22
- K**
- Keyboard
 - appearance..... 5
 - hide/show 15
 - Layouts..... 9
 - editing 13
 - placement 15
 - size..... 15
 - Keyboard Menu 6
 - Keys
 - color..... 17
 - font 17
 - size..... 16
- M**
- macro 30
 - commands 30
 - Menu Bar 5
 - Hide 16
 - Mouse Menu 6
- P**
- pointer
 - size..... 6
- R**
- Reboot 9
 - Run Menu..... 6
- S**
- Show Case..... 25
 - Status Bar 7
 - Hide 16
- T**
- text entry 25
 - capitalize 25
 - Title Bar 5
 - Hide 16
- V**
- View Menu 7, 15
- W**
- Word Bar..... 7, 9
 - Hide 16
 - Word List
 - edit 26, 27
 - statistics 28
 - Word Prediction..... 25



Origin Instruments Corporation
854 Greenview Drive
Grand Prairie, Texas 75050, USA
TEL: 972-606-8740 FAX: 972-606-8741
email: sales@orin.com Web: <http://www.orin.com>